



INSTRUCTIONS

Perfect-Patterns Face Mask Instructions:

Materials:

Outer fabric: 100% cotton such as quilt fabrics Lining and End Covers: 100% bleached muslin Filler: Same muslin as for lining, or thin quilt batting 1/8" flat elastic (cut 10" pieces) or make 36" tie strings from bias cut muslin One 12" pipe cleaner

Cutting Instructions:

Cut 2 outer fabric - 2 lining - 2 inner filler
Optional: Cut 2 filter
Cut 2 end covers
Cut two 9-11" pieces of elastic (smaller sizes take less)
(A thin tube turner helps for pulling elastic through casings.)

- 1. Right sides together layer: 1 inner filler 2 outer fabric 1 inner filler. Sew the curved notched edge from top to bottom using 1/4 inch seam. Trim the seam to 1/8 inch.
- 2. Right sides together sew the notched seam in the lining and trim.
- 3. Right sides together, sew the mask and lining together along the top (casing) edge and trim the seam.
- 4. Right sides together, sew the bottom edge together and trim the seam.
- 5. Turn the mask right side out and press.
- 6. Top stitch close to both the top and bottom edges.

Note: for the Small size cut the pipe-cleaner down to 10" before folding.

- 7. Sew a 3/8" casing along the top edge of the mask from one side to the other. Fold the pipe-cleaner in half and finger press the fold so it's tight. Thread the tube-turner through the casing and catch the fold of the pipe-cleaner pulling it into the center of the casing. Stitch the casing closed at each end of the pipe-cleaner.
- 8. Right sides together lay the end cover piece along the open end of the mask leaving at least 1/2" of fabric sticking out past each end. Sew the end cover to the mask.
- 9. With the end cover extended, turn in and press the two short ends of the cover. Turn in and press under the long edge. Turn the long pressed edge in to the line of sewing and press the outside fold. Sew the folded edge to the mask to form a casing.
- 10. Using the tube-turner pull a length of elastic trough the casing and tie the two ends of the elastic together in a tight compact knot. Pull the knot to hide it inside the casing.

If you prefer to make ties, cut two 36" lengths of muslin on the bias. Fold and press into a 1/4" double fold bias tape and sew along the open edge. Use the tube-turner to pull through the side casings.

SIZETAGS

REMOVE TAG	REMOVETAG	REMOVETAG	REMOVE TAG	REMOVE TAG	REMOVE TAG	REMOVE TAG	REMOVE TAC				
SMALL	MEDIUM	LARGE	X LARGE	SMALL	MEDIUM	LARGE	X LARGE	SMALL	MEDIUM	LARGE	X LARGE
	T										
REMOVE TAG	REMOVETAG	REMOVE TAG	REMOVE TAG	REMOVE TAG	REMOVE TAG	REMOVE TAG	REMOVE TAG				
SMALL	MEDIUM	LARGE	X LARGE	SMALL	MEDIUM	LARGE	X LARGE	SMALL	MEDIUM	LARGE	X LARGE

										7.	
											100
DEMO:							100				3 - A
REMOVE TAG	REMOVE TAG	REMOVE TAG			REMOVE TAG	REMOVE TAG	REMOVE TAG	REMOVE TAG	REMOVE TAG	REMOVE TAG	REMOVE TAG
SMALL	MEDIUM	LARGE	X LARGE	SMALL	MEDIUM	LARGE	X LARGE	SMALL	MEDIUM	LARGE	X LARGE
	1										
REMOVE TAG	REMOVE TAG	REMOVE TAG	REMOVE TAG	REMOVE TAG							
SMALL	MEDIUM	LARGE	X LARGE	SMALL	MEDIUM	LARGE	X LARGE	SMALL	MEDIUM	LARGE	X LARGE
DEMOVE TA	BENOVETA	DEMONETA -	BEUD T	BENOVETA							
REMOVE TAG			REMOVE TAG				REMOVE TAG			the state of the s	
SMALL	MEDIUM	LARGE	X LARGE	SMALL	MEDIUM	LARGE	X LARGE	SMALL	MEDIUM	LARGE	X LARGE
							- 1111111111111111111111111111111111111				
											S-12
REMOVE TAG	REMOVETAG	REMOVE TAG	REMOVE TAG	REMOVE TAG	REMOVE TAG	REMOVE TAG	REMOVE TAG				
					MEDIUM	LARGE	X LARGE	SMALL	MEDIUM	LARGE	
SMALL	MEDIUM	LARGE	X LARGE	SMALL	MEDIUM	((1 chicalo	0,000	11104210011	carego	X LARGE
	MEDIUM	LARGE	X LARGE	SMALL	MEDIUM		x critiqu	0,0,0	77104710071	CARCGO	X LARGE
	MEDIUM	LARGE	X LARGE	SMALL	MEDIUM		x critiqu	0,10 (00	77(042(007)	CARCYC	X LARGE
	МЕДІИМ	LARGE	X LARGE	SMALL	MEDIUM		x critical o	5,70 (80	71102001	LARGO	X LARGE